





CRYOABLATION OF THE KIDNEY

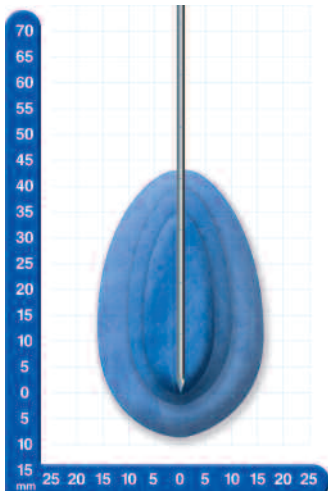


R1.7 RENAL CRYOPROBE
DIAMETER: 1.7mm SHAFT LENGTH: 15cm

	DIAMETER (MM)	LENGTH (MM)
ICEBALL	33	54
-20 C ISOTHERM	21	42
-40 C ISOTHERM	14	35

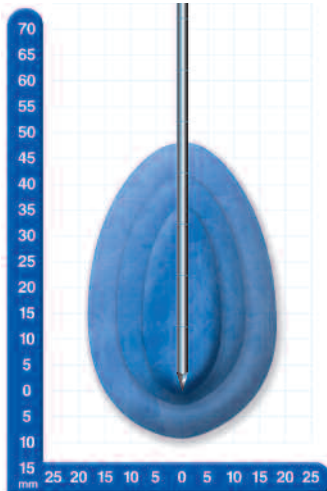

R2.4 & R2.4L RENAL CRYOPROBE
R2.4: DIAMETER: 2.4mm SHAFT LENGTH: 15cm
R2.4L: DIAMETER: 2.4mm SHAFT LENGTH: 23cm


	DIAMETER (MM)	LENGTH (MM)
ICEBALL	37	56
-20 C ISOTHERM	24	44
-40 C ISOTHERM	16	36

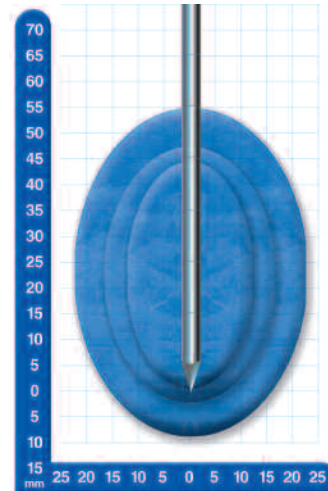

R3.8 & R3.8L RENAL CRYOPROBE
R3.8: DIAMETER: 3.8mm SHAFT LENGTH: 13cm
R3.8L: DIAMETER: 3.8mm SHAFT LENGTH: 28cm


	DIAMETER (MM)	LENGTH (MM)
ICEBALL	45	64
-20 C ISOTHERM	33	49
-40 C ISOTHERM	24	44

CRYOABLATION OF THE PROSTATE

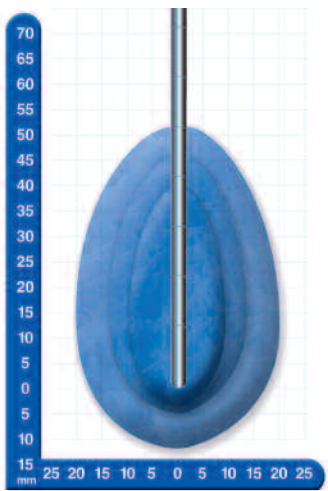


CRYO-40 CRYOPROBE
DIAMETER: 3.4mm SHAFT LENGTH: 23cm

	DIAMETER (MM)	LENGTH (MM)
ICEBALL	40	64
-20 C ISOTHERM	27	51
-40 C ISOTHERM	19	43

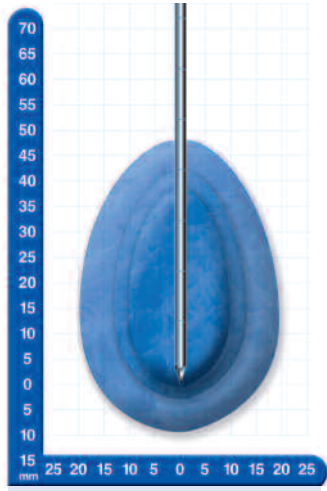

CRYO-44 CRYOPROBE
DIAMETER: 2.4mm SHAFT LENGTH: 15cm

	DIAMETER (MM)	LENGTH (MM)
ICEBALL	39	57
-20 C ISOTHERM	26	44
-40 C ISOTHERM	19	39

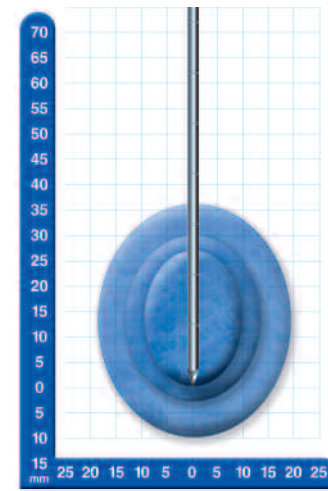

CRYO-2.4S CRYOPROBE
DIAMETER: 2.4mm SHAFT LENGTH: 15cm

	DIAMETER (MM)	LENGTH (MM)
ICEBALL	38	46
-20 C ISOTHERM	26	33
-40 C ISOTHERM	19	27

*DATA COLLECTED USING GELATIN FORMULA WHICH APPROXIMATES PERFORMANCE (±5MM) IN SOFT TISSUE AT 10 MINUTES.

THE RIGHT CRYOPROBE FOR THE RIGHT PROCEDURE.



TO LEARN MORE ABOUT ENDOCARE'S CRYOABLATION PRODUCTS
PLEASE CALL (800) 418-4677 OR VISIT WWW.ENDOCARE.COM